


Operation methods

Most of operations can be done by touching the screen directly. You can also operate with gloves or by using the tip of a stylus.

Tips



The perception system of the touch panel used for the EyeSpecial C-III is different from the ones of smart phones. Gently touch the screen with your finger nails or tip of your finger.

You can also operate this camera with function keys (small silver buttons) on the camera frame outside of the touch screen. For more details, refer to page 10 in the User's Manual.

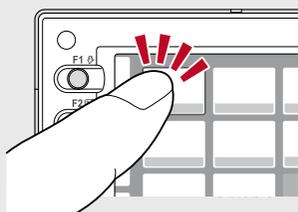

Turn the camera on

Rotate the power switch to turn the camera on.

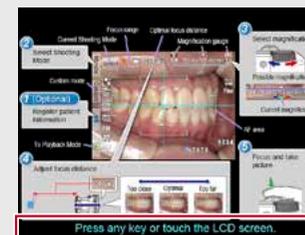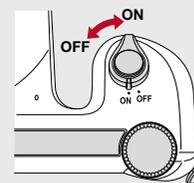

When the start screen appears on the display, press any key or touch the LCD screen and you can start taking pictures.

When you turn the camera on for the first time, you may need to set the language, date, time and units. For more details, refer to page 16/17 in the User's Manual.

Taking pictures


Follow steps 1 - 5 to take pictures

1 Input of patient information (optional)

For more details, refer to page 20 in the User's Manual.

2 Select the shooting mode

Press **[Mode]** to display all shooting modes preinstalled. Select the optimal one for each purpose.

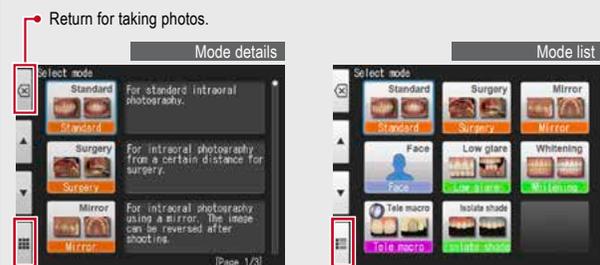

Press here to alternate between mode details and mode list.

For more details on each shooting mode, refer to the back of this Quick Reference.



Learn more about formatting SD memory cards on page 36 in the User's Manual.

Should you inadvertently press any keys while shooting and a different menu appears, half-press the shutter button to return to the shooting mode.

You can attach the patient ID to the pictures or you can register your custom settings in the camera. For more details, please refer to page 20 in the User's Manual.


3 Select the magnification ratio

Adjust the magnification ratio by rotating the dial key. (You can't change the magnification ratio by touching the screen.)

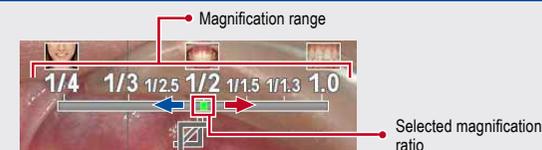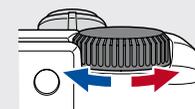

4 Set the focus distance

Based on the optimal focus distance displayed on the upper area of the screen, adjust the distance between the camera and the subject. Change the camera position until the subject appears on the screen sharply.

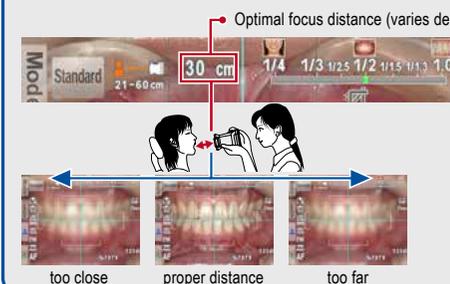


Tips Taking pictures in Face mode

When the "▲ UP" icon appears on the screen, hold the camera as indicated by the ▲ icon.



5 Focus and take pictures

Half-press the shutter button to focus. When properly focused, the green focus indicator appears on the right bottom of the screen. Then, press the shutter button completely down to capture the photo.

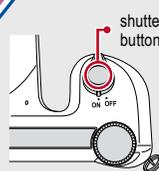

Focus indicator ●: in focus
●: out of focus

Tips Spot Auto-focus

You can focus the area outside of the Auto-focus area by touching the area you wish to focus directly on the screen (Spot AF mode). For returning to the standard AF mode, keep pressing the spot-focused area for a few seconds.



